

TREATMENT OF MINOR SKIN IRREGULARITIES WITH THE CLINICAL SKIN CLEAR



The Clinical Skin Clear can be used to minimize the appearance of:

- Hyperpigmentation • Skin Tags • Spider Naevi
- Cholesterol Deposits • Milia • Telangiectasis
- Acne Pimples • Fibroma Simplex
- Clogged Pores • Broken Capillaries

How it works:

The Clinical Skin Clear System works on the principle of radio-vapourization. This technology utilizes a combination of high frequency and radio frequency, which are attracted to liquids on the surface of the skin or within the skin irregularity. This enables the therapist to treat unsightly skin abnormalities without penetrating the skin's surface.

Treatments are quick [often within seconds of application process] and the treatment is tolerable. Treatment times and costs vary depending on the type and size of the skin lesion being treated. Multiple treatments or longer treatment times may be required to effectively reduce the appearance of the irregularity completely without obvious scarring.

There is minimal discomfort and healing time is short, from a day or two for skin tags to a few weeks for larger skin abnormalities such as fibromas, cholesterol deposits or milia.



Telangiectasis



Fibroma on Face



Fibroma on Neck



Lesion on Lips



Skin Tag with Stalk



Flat Skin Tag



Cholesterol Deposits



Large Fibroma



Acne Pimples



Clogged Pores



Raised Angioma



Cherry Angiomas



Skin Tag Before



Skin Tag After



Cholesterol Deposit
Before



Cholesterol Deposit
After